VERMEIREN

268 - 269

INSTRUCTION MANUAL

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, March 2012

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

© N.V. Vermeiren N.V. 2012

EN Contents

1. Technical details.....	2
2. Components	2
3. Explanation of symbols	3
4. Check after delivery.....	3
5. Intended use.....	3
6. Fold and unfold the rollator.....	3
7. Assembly.....	4
8. Height adjustment handgrips	4
9. Brakes	5
10. Safety instructions	5
11. Care.....	5
12. Disinfection.....	5
13. Repairs / Service / Inspection.....	5
14. Shipping and storage	6
15. Warranty.....	6
16. Disposal.....	6
17. Declaration of conformity.....	6

MANUAL

4-wheels walking tables 268, 268 Junior and 269

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Walking table
Model	268
Width	580 mm
Length	760 mm
Height	1160 mm - 1460 mm
Width (folded)	125 mm
Height (folded, without handgrips)	860 mm
Weight	15,5 kg
Height adjustment table	990 mm - 1290 mm
2 Front wheels	Ø 75 mm
2 Rear wheels	Ø 75 mm with brakes
Max. Turning width	920 mm
Max. inclination	use on flat underground
Maximum user weight	max. 120 kg
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Walking table
Model	268 Junior
Width	580 mm
Length (unfolded)	685 mm
Height	985 mm - 1284 mm
Width (folded)	125 mm
Height (folded, without handgrips)	684 mm
Weight	15 kg
Height adjustment table	815 mm - 955 mm
2 Front wheels	Ø 75 mm
2 Rear wheels	Ø 75 mm with brakes
Max. Turning width	850 mm
Max. inclination	use on flat underground
Maximum user weight	max. 120 kg
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Walking table with arm supports
Model	269
Width	580 mm
Length	760 mm
Height	1160 mm - 1460 mm
Width (folded)	125 mm
Height (folded, without handgrips)	860 mm
Weight	12,5 kg
Height adjustment arm supports	975 mm - 1275 mm
2 Front wheels	Ø 75 mm
2 Rear wheels	Ø 75 mm with brakes
Max. Turning width	920 mm
Max. inclination	use on flat underground
Maximum user weight	max. 120 kg
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg	

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

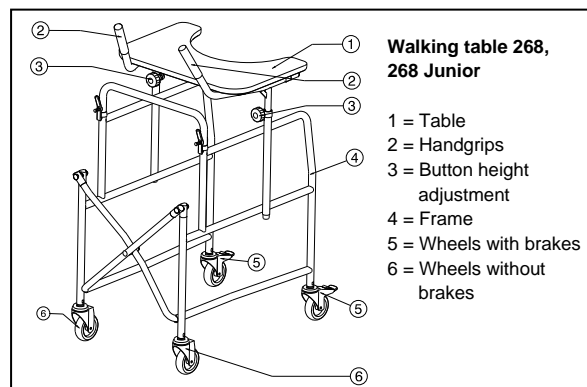
Before using the walking table, read the instruction manual carefully: it will familiarise you with this product.

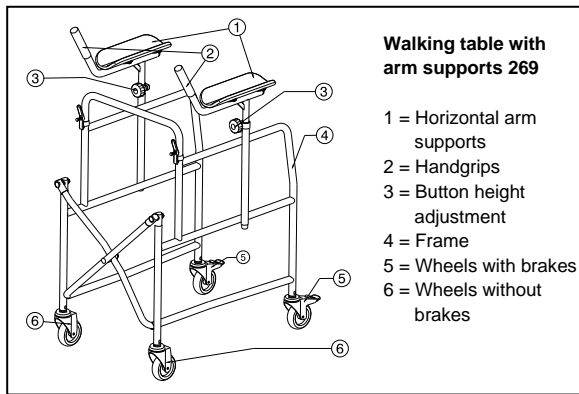
You will also be given hints about the care and service of your walking table. Please remember that observance of our hints would keep your walking table in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

2. Components

3. Explanation of symbols

Maximum mass

Only for indoor use

CE conformity

Read the instruction manual before use

Observe the safety instructions

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- frame walking table, 2 wheels without brakes (mounted) and 2 wheels with brakes (mounted)
- table (for 268 or 268 Junior) or horizontal arm supports (for 269)
- handgrips
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

The walking table will compensate the lack of adequate support by the legs by leaning on, or moving with the walking table, with usage of the upper limbs or chest. The walking table will provide a better stability, support and safety during walking.

The walking table is only for indoor use, for 1 person.

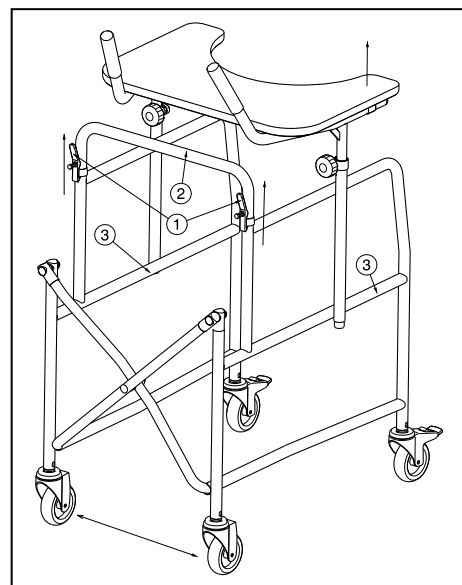
The product is not meant for transport of goods.

6. Fold and unfold the walking table

⚠ WARNING: Risk of clamping - Keep fingers, buckles and clothes away from the folding mechanism.


The walking table can be folded according following instructions:

- Remove the table or horizontal arm supports. With the four clips just below the table you can also remove the table from the handgrip tubes.
- Press on the levers ①.
- Remove the horizontal frame tube ②.
- Pull the tubes ③ to each other to fold the walking table.

The walking table can be unfolded according following instructions:

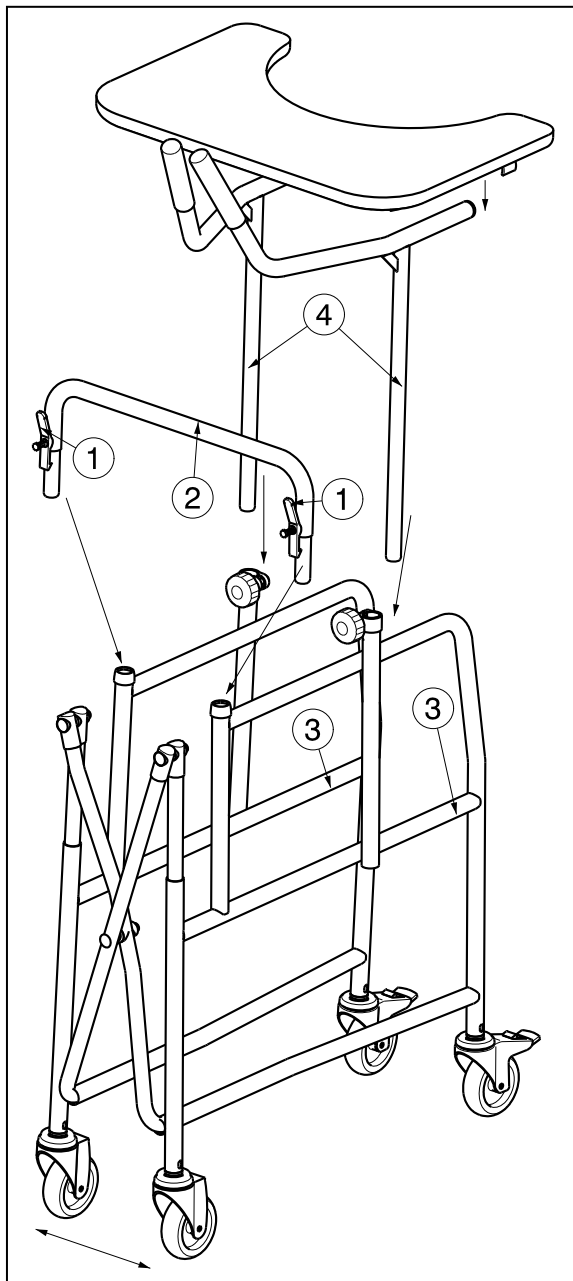- Unfold the walking table by pulling the tubes ③ outwards.
- Mount the horizontal tube ②.
- Check that the levers ① are well fixated.
- Mount the handgrip tubes ④ or horizontal arm supports. Check that these are well fixated. The table can be mounted with the four clips on the handgrip tubes ④.

7. Assembly

- Unfold the walking table frame.
- Mount the table (for 268, 268 Junior) or the horizontal arm supports (for 269).
- Adjust the handgrips to a comfortable height.

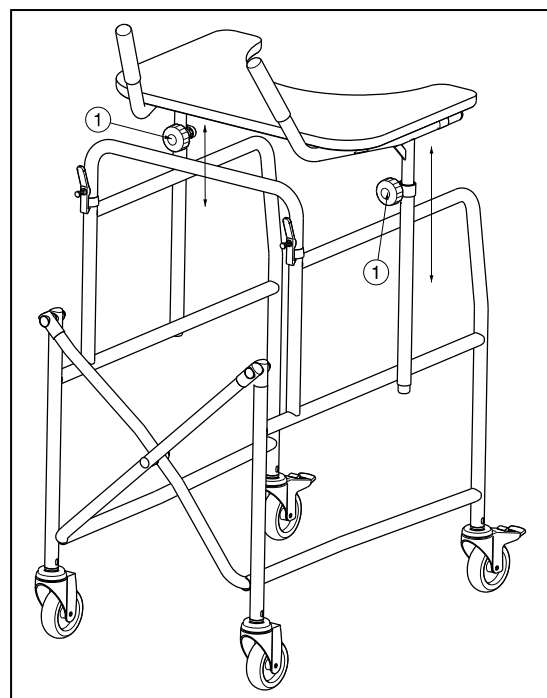
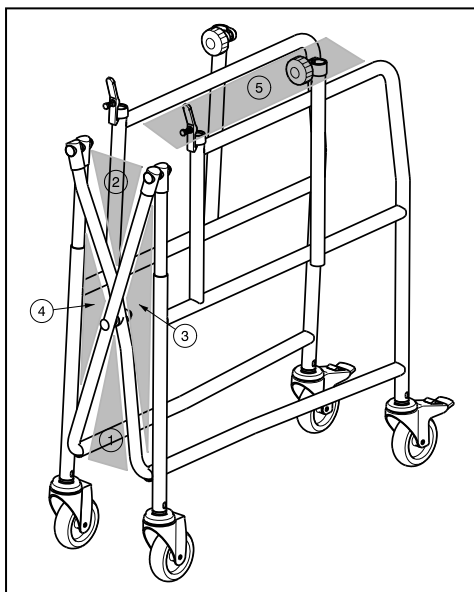
8. Height adjustment handgrips

- ⚠ **CAUTION:** Risk of falling - Firmly fix the handgrips before using the walking table.
- ⚠ **CAUTION:** Risk of falling - Do not exceed the maximum adjustment, your walking table will not be stable anymore.
- ⚠ **CAUTION:** Use the handgrips only parallel to the walking direction of your walking table.

The handgrips + table (268) or handgrips + horizontal arm supports (269) can be adjusted in height.

- Loosen the buttons ① on both sides **simultaneously** for **268 walking table** or on each side **separate** for **269 walking table**.
- Adjust the height of the handgrips to a comfortable position. When standing straight, the support should be as high that you can easily lay your arms on the support. **The minimum and maximum adjustment is indicated by the anti-slip zone on the tube (range of 300 mm).**
- Lock the buttons ① firmly by hand.


The 5 zones for risk of clamping are indicated in the figure below:

9. Brakes


- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the wheels (water, oil, mud, ...) - Check the condition of the wheels before each use.
- ⚠ **WARNING:** The brakes can wear - Check the operation of the brakes before each use.
- ⚠ **WARNING:** Risk of unintended movement - Make sure the walking table is on a flat horizontal surface before releasing the brakes.

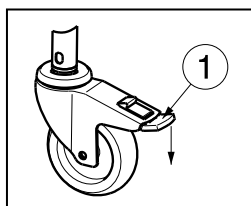The walking tables are equipped with two parking brakes on the rear wheels.

Engaging the parking brakes

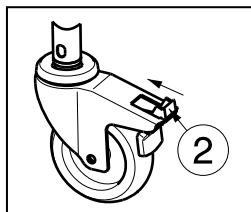With your foot, press the brake plate ① downwards till it locks in place.

Disengaging the parking brakes

With your foot, press the brake plate ② forwards in the direction of the frame.

10. Safety instructions

- Ensure there are no stones or any other objects blocking the regular usage of the wheels of the walking table.
- Ensure the brakes or the walking tables are always in parking-mode before using your walking table. When you are standing comfortable and stable, you may unlock the parking brakes.
- Before every usage, check that the buttons for height adjustment of your table are firmly locked.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your walking table when you are standing comfortably and when both your arms are resting on the table/arm supports.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- **The maximum users weight for your walking table is 120 kg** and can only be used on a flat underground.

- Do not use your walking table on a staircase or bumps higher than 40 mm.
- Avoid undergrounds with a very steep inclination.
- Do not use your walking table to transport goods.
- Risk of injury - Surfaces can assume the environment temperatures.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

11. Care

Use only a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your walking table is preserved in a perfectly functional condition.

12. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

13. Repairs / Service / Inspection

Lifetime of the walking table is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the walking table, it should be serviced according to the following criteria:

- Completeness
- Adjustment functions
- Check the casters (stability, tracking, fixing screws)
- Check the condition of the brakes on the rear casters

- Setting of the adjustment buttons (condition, thread, secure locking)
- Frames (deformation, stability, connections)
- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)
- Check folding mechanism
- Check height adjusting system

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review
- Disinfection

14. Shipping and storage

The shipping and storage of the walking tables shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the walking table from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The walking table must be stored without being subjected to strains. (Do not put too heavy parts on the walking table, not clamping between something, ...)

15. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The guarantee period for warranty claims is 24 months.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

16. Disposal

When disposing of the walking tables, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

17. Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15
2920 Kalmthout
Belgium

declares under his sole responsibility that the CE marked medical devices :

Productgroup: Rolling crutches with arm supports
Brand: Vermeiren
Type: 268, 269, 276

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim this warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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Dealer's stamp:

Date:

- For service checklists an additional technical information, please see our specialist dealers nearest to you. More information on our website at: www.vermeiren.com.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Reatime S.R.L.

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.reatime.it
e-mail: info@reatime.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Spain

Vermeiren Iberica, S.L.

Trens Petits, 6. - Pol. Ind. Mas Xirgu.
17005 Girona
Tel: +34 902 48 72 72
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

L. Vermeiren Ges. mbH

Winetzhammerstraße 10
A-4030 Linz
Tel: +43(0)732 37 13 66
Fax: +43(0)732 37 13 69
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Hühnerhubelstraße 59
CH-3123 Belp
Tel: +41(0)31 818 40 95
Fax: +41(0)31 818 40 98
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

The Netherlands

Vermeiren Nederland B.V.

Domstraat 50
NL-3864 PR Nijkerkerveen
Tel: +31(0)33 2536424
Fax: +31(0)33 2536517
website: www.vermeiren.com
e-mail: info@vermeiren.be

Czech Republic

Vermeiren ČR S.R.O.

Sezemická 2757/2 - VGP Park
193 00 Praha 9 - Horní Počernice
Tel: +420 731 653 639
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz