HP0180W LITHIUM BATTERY CHARGER INSTRUCTIONS

Operating Instruction:

- (1) Make sure the power cord, battery cable, battery terminals are in good condition.
- (2) Make sure the charge output voltage is the same as the connecting battery.
- (3) Connect the connector of battery charger to the connector of battery firstly, then plug the AC plug into the electricity outlet.

LED Indication:

Green light on: Power on, disconnect with the battery

Orange light on: Charging

Green light on: Fully-charged

Trouble Shooting

- (1) Green light is off when power on.
Check if the input power cord of the charger has been plug into the socket, and if there have electricity passes from the outlet, if yes; please send the charger back to the manufacture for repair
- (2) Orange light is off when connect with the battery.
Check if the charger and battery connectors are correctly connected, if both of them are in good connection, please send the battery for repair because it can not be charged.
- (3) Orange light turns to Green immediately.
Check if the battery is fully charged, if not, the charger maybe defective, please send the charger back to the manufacture for repair

Caution:

- (1) For indoor use only, do not expose to rain.
- (2) Use the charger only with less than 36V 40AH li-ion batteries.
- (3) For safety consideration, please disconnect the input side before removing the charger from the battery.
- (4) Keep the charger away from explosive gas and sparks and put it in a well ventilated area during charging.
- (5) To prolong the life of the charger, please do not connect the charger and battery for a long time after full-charged.

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