# Installation manual

MANUEL D'INSTALLATION  
INSTALLATIEHANDLEIDING  
INSTALLATIONSANLEITUNG  
MANUALE DI INSTALLAZIONE  
MANUAL DE INSTALACIÓN  
INSTRUKCJA INSTALACJI  
PŘÍRUČKU PRO INSTALACI

## Turios

# Contents

|                                        |          |
|----------------------------------------|----------|
| <b>Preface</b>                         | <b>4</b> |
| <b>1. This product</b>                 | <b>5</b> |
| <b>2. Scope of delivery</b>            | <b>6</b> |
| <b>3. Assembly and adjustments</b>     | <b>7</b> |
| 3.1. Tools                             | 7        |
| 3.2. Seat inclination (manual version) | 8        |
| 3.3. Seat height (optional)            | 8        |
| 3.4. Seat depth                        | 9        |
| 3.5. Width between armrests            | 11       |
| 3.6. Armrest height                    | 12       |
| 3.7. Armpad depth                      | 12       |
| 3.8. Operating console                 | 13       |
| 3.9. Footrest length                   | 14       |
| 3.10. Footplate position               | 14       |
| 3.11. Footplate angle                  | 15       |
| 3.12. Adjusting the manual brakes      | 15       |
| 3.13. Changing the battery             | 16       |
| 3.14. Changing wheels or tyres         | 18       |

# Preface

To support you on the installation and repairs of this powered wheelchair, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following wheelchair(s):

- Turios

## Important note

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

## Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

|                                                                                     |                                                                                                 |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------|
|  | User manual<br>For user and specialist dealer                                                   |
|  | User manuals of applied operating console and battery charger<br>For user and specialist dealer |
|  | Installation instructions<br>For specialist dealer                                              |
|  | Service manual for wheelchairs<br>For specialist dealer                                         |
|  | Drawings of (spare) parts<br>For specialist dealer                                              |

# 1. This product

1. Backrest
2. Armrests
3. Arm pads
4. Operating console
5. Seat
6. Safety belt
7. Leg rests or footrests
8. Footplates
9. Steering wheels (front wheels)
10. Driving wheels (rear wheels)
11. Driving motors
12. Battery housing

13. Front light (optional)
14. Rear light (optional)
15. Anti-tipping
16. Identification plate

## 2. Scope of delivery

The following components are part of the delivery:

- Framework with operator control, rear and front wheels, seat and backrest
- Armrests
- Footrests
- Batteries
- Battery charger
- Tools
- User manual of:
  - wheelchair
  - Operating console
  - Battery charger
- Options

Before use, check if everything is included and that no products are damaged.

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

## 3. Assembly and adjustments

### CAUTION

#### **Risk of injuries or damage**

- Only use the settings described in this manual.
  - Only use parts described in this manual.
  - Only use genuine spare or replacement parts approved by Vermeiren.
  - Prevent bystanders from accessing the working area.
  - Keep the (dis)assembling area clean and tidy, to prevent falling.
  - Collect all small parts in a box or bag.
  - While (dis)assembling and operating the wheelchair, make sure no objects or body parts get caught between moving parts. In particular, pay attention to the position of the cables.
  - Turn the wheelchair off before you perform any mechanical adjustments and make sure that the free-wheel lever is in braking position.
  - Make sure all screws are firmly secured before driving.
  - Software changes are only allowed if executed by skilled personnel, taking into account the user and environment (stability, acceleration, indoor/outdoor, etc.).
-  Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

### 3.1. Tools

To (dis)assemble and adjust the wheelchair, the following tools are needed:

- Wrench no. 10, 13, 17
- Allen key no. 4, 5, 6
- Screwdriver

## 3.2. Seat inclination (manual version)


### CAUTION

#### Risk of injuries or damage

- The frame with seat is heavy to handle. Ask help of an assistant.

The seat angle can be manually adjusted according to the following steps:

1. Remove the footrests.
2. Remove the locking pin (1) that connects the seat and inclination rod.
3. Slide the outer tube (3) up/down to the desired angle (see table).
4. Replace the locking pin (1) and close securely.
5. Replace the footrests.

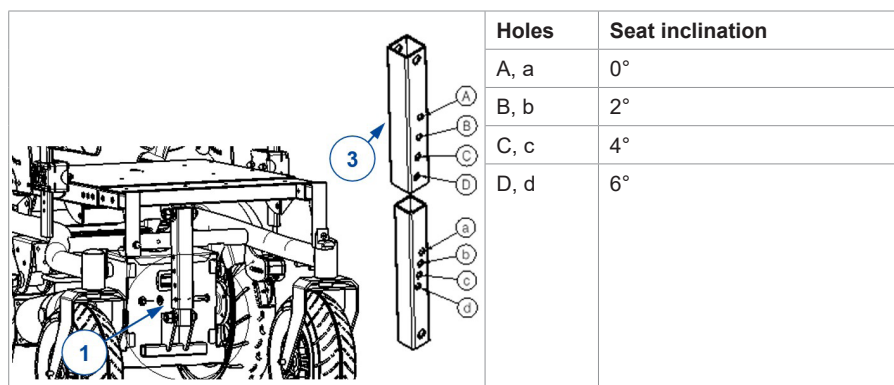
## 3.3. Seat height (optional)

The seat height can be raised by placing an additional frame section between the lower frame and seat frame.

1. Remove the bolt-nut connections that connect the seat frame with the lower frame [A] and seat inclination mechanism/actuator [B].
2. To raise the seat
  - by 50 mm (manual) / 30 mm (electrical), use part (4).
  - by 100 mm (manual) / 60 mm (electrical), use parts (4) and (5).

3. Firmly tighten all screws connecting the seat frame, additional frame section and lower frame.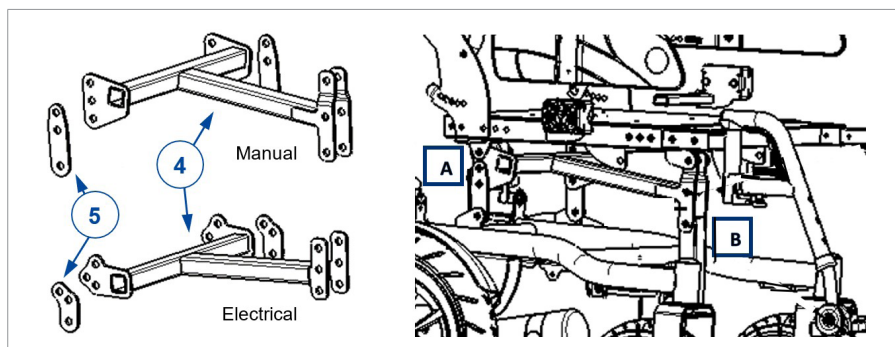

### 3.4. Seat depth

#### 3.4.1. Manual version

To adjust the seat depth, you can adjust the position of the backrest (60 mm, steps of 20 mm) and the position of the front frame with footrest supports (60 mm, steps of 20 mm). It is recommended to start by adjusting the backrest.


To adjust the backrest:

1. Remove the screws (6) on the side of the frame.
2. Slide the backrest attachment to the front/back until the desired position.
3. Replace both screws (6) and tighten firmly.
4. Make sure that both sides of the backrest are positioned identically.

To adjust the front frame with footrest supports:

1. Remove the footrests.
2. Remove the screws (7) on the side of the frame.
3. Remove screws (8) and loosen screws (9) from the seating plate.
4. Slide the inner tube forward/backward until the desired position.


5. Replace and firmly tighten all screws (7+8+9). Make sure that both sides of the front frame are positioned identically.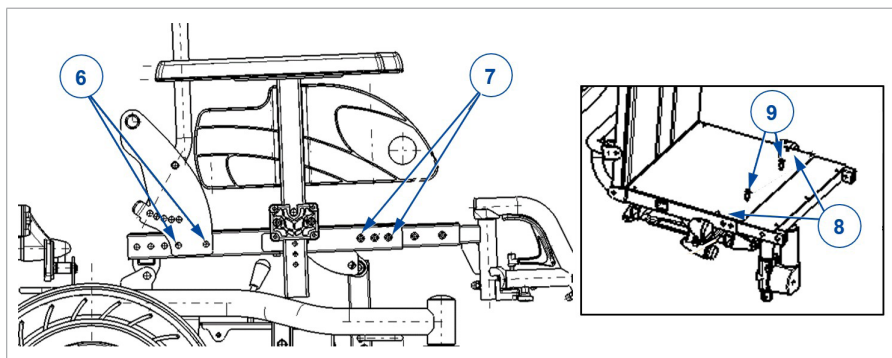

### 3.4.2. Electrical version

To adjust the seat depth (120mm, steps of 20mm):


1. Remove the screws (10) on the side of the frame.
2. Loosen screws (9) from the seating plate.
3. Slide the inner tube forward/backward until the desired position.
4. Replace and retighten the screws (9+10).
5. Make sure that both sides are positioned identically.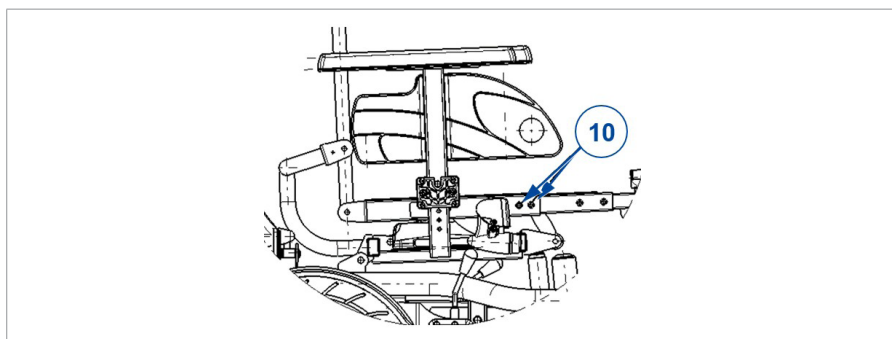

### 3.5. Width between armrests

1. Remove the armrest.
2. Loosen the screw (11) under the seat frame until the armrest support can be moved.
3. Slide the armrest support inwards/outwards in the support tube until the desired position (range 60 mm: step less).
4. Retighten the screw (11) properly.
5. Repeat instructions above for the other arm support.


Check that the arm supports are positioned identically on both sides and that they are fixed securely.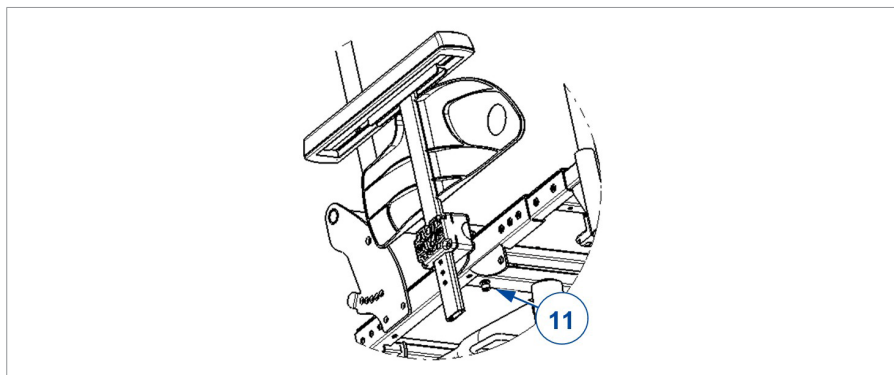

### 3.6. Armrest height

The height of the armrest is adjustable in 6 positions as follows:


1. Pull the armrest out of the armrest holder.
2. Remove the socket screw (36) that sets the height.
3. Mount the screw in the desired hole.
4. Insert the armrest in the armrest holder.
5. Check the height; readjust if necessary.
6. Repeat this for the second armrest. Make sure both armrests are positioned identically.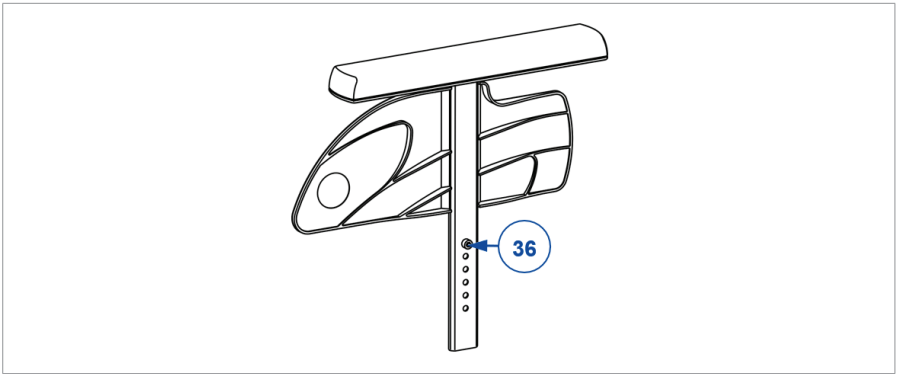

### 3.7. Armpad depth

Adjust the depth of the arm pad as follow (range 120 mm: step less):

1. Loosen both socket screws (35) at the bottom of the arm pad.
2. Move the arm pad to the front or rear within the range of the slotted holes (42,5 mm).  
If this not enough, remove the socket screws and attach the arm pad using the other holes (3 positions available).
3. Tighten both socket screws firmly.
4. Repeat this for the second arm pad. Make sure both arm pads are positioned identically.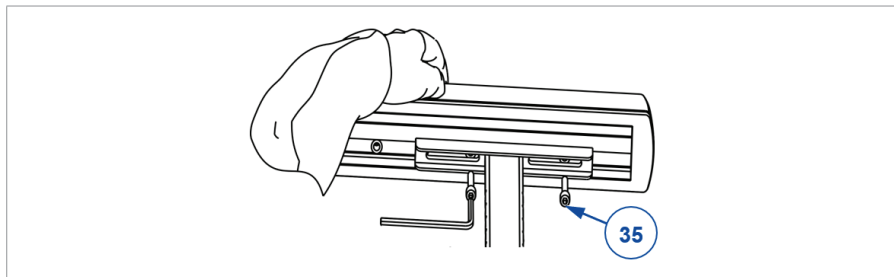

## 3.8. Operating console

### 3.8.1. Operating console depth

The operating console can be adjusted in depth:

1. Loosen the screws (35) under the armpad.
2. Slide the operating console support to the front/back until the desired position.
3. Retighten the screws (35).

### 3.8.2. Mounting position left or right

1. Loosen and remove the screws (35) under the armpad.
2. Take off the armpad and operating console support. Replace the armpad and fasten in place by tightening the two screws (35).
3. Loosen and remove the screws (35) under the other armpad.
4. Take off the armpad, place the operating console support and replace the armpad. Fasten everything into place by tightening the two screws (35).

### 3.9. Footrest length

#### **⚠ CAUTION**

#### **Risk of damage**


- Make sure that the footrests won't touch the ground. Keep a minimum safe distance of 60 mm above the ground.

Adjust the length of the footrests as follows (3 possible positions on outer tube, steps of 35 mm ; 6 possible positions on inner tube, steps of 25 mm):

1. Remove the socket screw (26) at the back of the footrest.
2. Adjust the footrest to a comfortable length.
3. Retighten the socket screw firmly.
4. Repeat this for the second footrest.  
Make sure the footrests are positioned identically.

### 3.10. Footplate position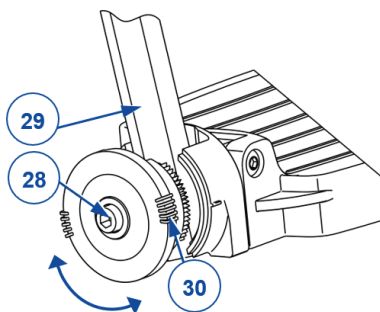

The footplate(s) can be used in 2 positions: front and rear. By default, each footplate is adjusted with its base plate in the rear position.

To place the footplate in front position:

1. Remove the socket screws (28) of both footplates.
2. Exchange left and right footplate.
3. Tighten the socket screws (28) firmly.

### 3.11. Footplate angle

Adjust the angle of the footplate at its toothed clamp:

1. Loosen the socket screw (28) of the footrest (29).
2. Turn the footplate to the desired angle (80° - 100°). The dashes (30) on the toothed clamp indicate the angular positions.
3. Retighten the socket screw.
4. Repeat this for the second footplate.

### 3.12. Adjusting the manual brakes


#### CAUTION

##### **Risk of injuries**

- Always adjust the parking brakes after adjusting the rear wheel position. Readjust also when tyres are worn, to compensate the wear.

Adjust the correct braking force.

1. Release the brake lever (1).
2. Loosen socket screws (22) to make the brake free to slide.
3. Slide the brake mechanism to the desired position. The adjustment range is determined by the slots in the mounting plate.
4. Retighten the socket screws (22).
5. Verify if the brake is working properly: the wheel is blocked, but the brake can be operated easily. If this is not the case, repeat the steps above.
6. Repeat this adjustment for the second parking brake. Make sure both brakes are adjusted identically.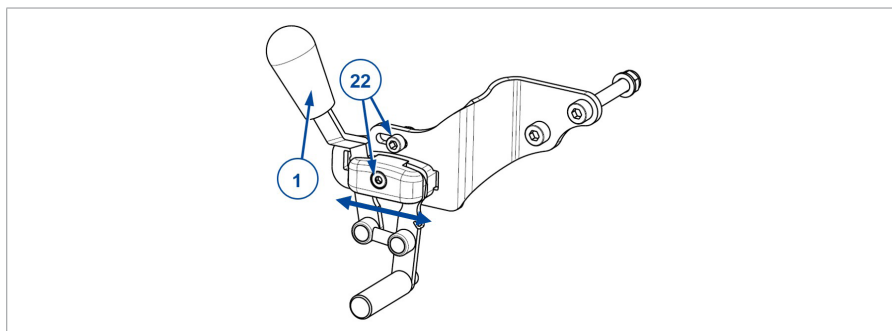

### 3.13. Changing the battery

#### **⚠ CAUTION**

#### **Risk of injuries or damage**


- The batteries must only be replaced by trained personnel.
- Do not come into contact with the acid from the batteries.
- Replace the batteries in a well-ventilated room.
- Make sure that the ventilation holes of the battery housing are free from any contaminations.
- First read the charging instructions in the user manual before charging the batteries.
- Do not place fingers between the batteries and the battery housing.

**i** Replace both batteries at the same time and with the same type as the ones used in the wheelchair. If this is not possible, contact the nearest Vermeiren facility.

1. Switch OFF the wheelchair.
2. Remove the screw (18) on the top, and the two screws (19) on the bottom of the plastic battery box cover (20).
3. Remove the plastic cover.
4. Untie the strap (21) securing the battery.
5. Pull the inner drawer (22) with batteries (23) out of the battery box.
6. Disconnect the batteries, see §3.13.1.. Remove them from the drawer.
7. Replace with the new batteries and connect them according to §3.13.1..

8. Carefully slide the inner drawer with batteries back into the battery box. Make sure that no wires are clamped between the batteries, drawer or battery box.
9. Tie the battery securing strap (21).
10. Place the battery box cover (20) and firmly tighten the two screws (19).
11. Replace the screw on top (18) and tighten firmly.
12. Switch ON the wheelchair to check if it works properly.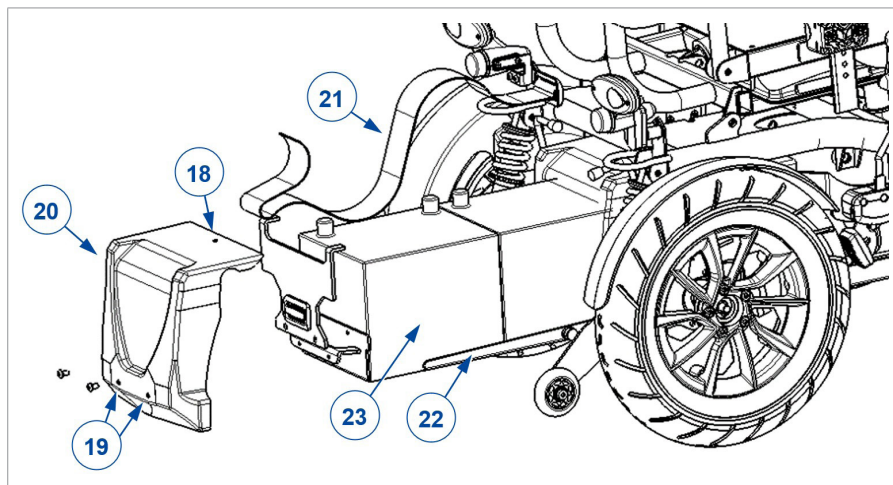

### 3.13.1. Battery connections

#### **⚠ CAUTION**


#### **Risk of injuries or damage**

- Check that no tools or other conductive objects come between the batteries poles, to avoid unintentionally current flows.
- Do not work in a humid environment with batteries and electronics.

The best way to disconnect the batteries:

1. Disconnect the main negative pole (-).
2. Disconnect the positive pole (+).
3. Disconnect the bridge.

To connect the batteries, use the reverse order.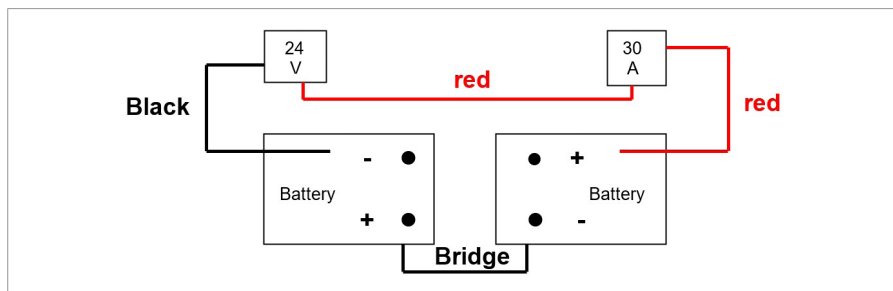

### 3.14. Changing wheels or tyres

#### **⚠ CAUTION**


#### **Risk of injuries or damage**

- Make sure that all air is released from the tyre, before removing it.
  - Use a wooden block and jack to lift the wheelchair. Always place the jack under the lower frame, never under any other parts.
  - Make sure that no body parts, the inner tube nor any other objects are pinched between tyre and rim.
  - Handle carefully to prevent damage to the rim.
1. Replace the inner tube, and if necessary also the tyre.
  2. Inflate the tyre using inflating equipment which complies to regulations and with a pressure indication in “bar”.
  3. Make sure that the pressure is correct, see the pressure indication on the tyres.

### 3.14.1. Driving wheels

**i** If a tyre needs to be replaced, first check the type of tyre that has been fitted. In case of a tyre with line profile, do not switch left and right tyre.

1. Lift the driving wheel from the ground using a jack.
2. Remove the socket screws (24) and take the wheel from the flange.
3. Open tyre valve and push it gently to release the air.
4. Loosen the 5 screws that connect both rim halves (on the inner rim half).
5. Separate the rim halves and remove the inner tube from the tyre.
6. Inflate the new or repaired inner tube partly.
7. Insert the inner tube into the tyre.
8. Place the two rim halves back on the tyre. Make sure that the tube valve pops out of the opening in the rim.
9. Connect the rim halves with the 5 socket screws. Tighten by hand.
10. Inflate the tyre at the correct pressure, see the indication on the tyre.
11. Remove grease and other contamination from the thread of the 5 flange socket screws. Apply screw adhesive (ex. Loctite) to these screws.
12. Mount the wheel on the flange.
13. Lower the jack.
14. Make sure the socket bolts are firmly secured.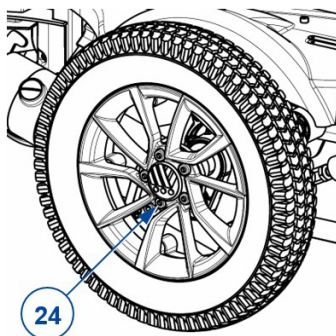


### 3.14.2.Steering wheels

#### 3.14.2.1.Dismantling

1. Loosen the screw on the steering wheel axle (25) and remove it from the steering wheel fork.
2. Let the air out of the steering wheel by lightly pressing the pressure pin on the valve.
3. Loosen the 5 screws that hold the split rim together (26).
4. Now separate the rim sides.

#### 3.14.2.2.Assembly

1. Insert the partly-filled inner tube into the tyre.
2. Connect the two sides of the rim through the tyres and use the 5 connecting screws to screw the rim together (26).
3. Make sure that the valve juts out of the valve opening in the rim.
4. Put the wheel back into the front wheel fork (25) and inflate the wheel.

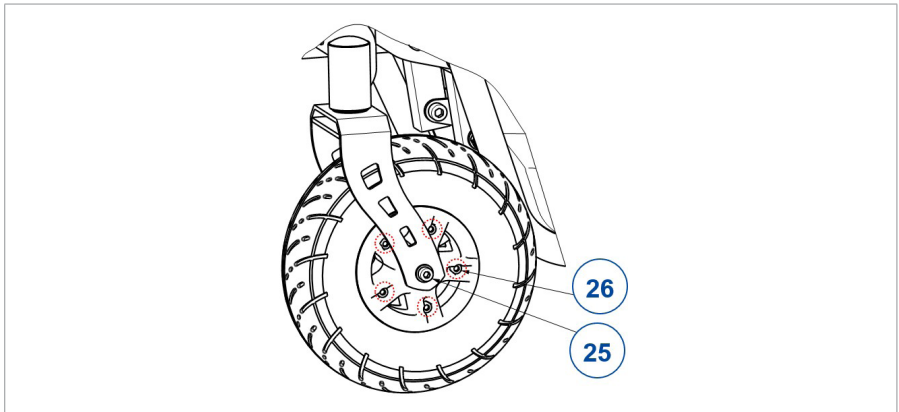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All rights reserved, including translation.