# VERMEIREN

## Electra

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

EN

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**EN Contents**

1. Technical details .....	2
2. Components .....	2
3. Explanation of symbols .....	2
4. Check after delivery .....	2
5. Intended use / environment .....	2
6. Placing / Removing batteries .....	2
7. Operating LED light and warning light .....	3
8. Operating the buzzer .....	3
9. Angle adjustment LED light .....	3
10. Height adjustment walking stick .....	3
11. Walking .....	4
12. Walking on stairs .....	4
13. Safety instructions .....	4
14. Maintenance .....	4

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Walking stick Electra

**1. Technical details**

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

<b>Make</b>	Vermeiren	
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout	
<b>Type</b>	Walking stick	
<b>Model</b>	Electra	
Width	160 mm	
Total height	775 mm – 1000 mm	
Height handgrip	765 mm – 990 mm	
Depth	39 mm	
Handgrip length	120 mm	
Weight	0,35 kg	
Maximum user weight*	max. <b>120 kg</b>	
Batteries	AAA, 1,5 V (2 batteries)	
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg.		

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the walking stick, read the instruction manual carefully: it will familiarise you with this product.


Refer to the Vermeiren website for the instructions for the care and safety of the walking stick. Please remember that observance of our hints would keep your walking stick in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

**2. Components**

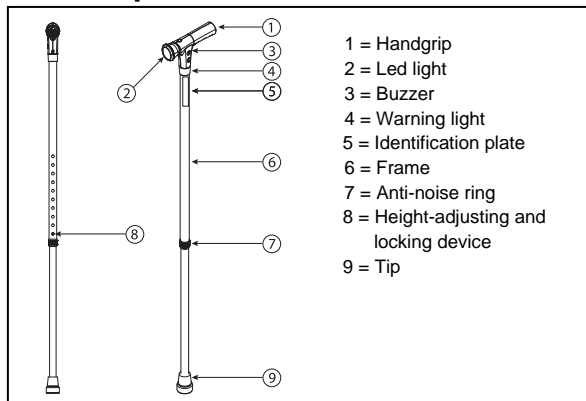**3. Explanation of symbols**

Maximum mass

CE conformity

**4. Check after delivery**

Unpack your product and verify if the delivery is complete. Following items should be included:

- 1 walking stick + Handgrip with LED light, warning light
- Tip
- Height adjustment system
- Manual

The two AAA batteries of 1,5 V are not included with the delivery.

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

**5. Intended use / environment**

**⚠ WARNING: Risk of falling - Do not use the walking stick if you have weak arm, wrist joint function or poor equilibrium.**

A walking stick give the user support during walking and makes it easier to walk. These support can be useful for balance problems or pain when walking.

The walking stick can be used indoor and outdoor.

Be extra careful that you do not slip when using on wet surfaces.

However, it cannot be used by persons with a visible weakened hand-/arm function, wrist joints.


The walking stick is not suited to use in high mountains, or on rough underground (big stones, ...).

**6. Placing / Removing batteries**

To place the batteries use instructions below:

- Open the flap ① on the backside of the handgrip ②, by moving the locking mechanism upwards.
- Remove first the old batteries (if available).
- Place the new batteries in the direction like shown in the figure below.


- Close the flap ① on the backside of the handgrip ②, by moving the locking mechanism downwards.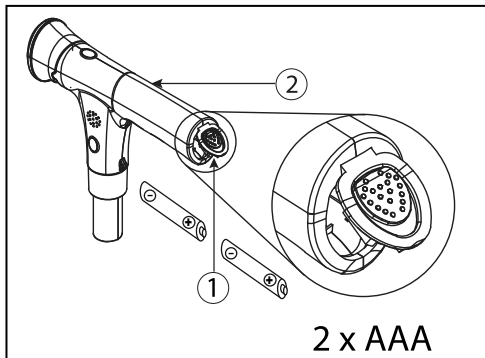

## 7. Operating LED light and warning light

To operating the LED light and warning light use instructions below:

- Press once on the button ①, the LED light lit.
- Press again or twice on the button ①, the LED light and warning light lit at the same time.
- Press again or three times on the button ①, the LED light and warning light goes out.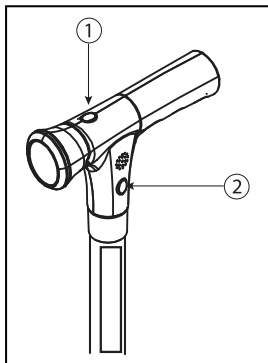

## 8. Operating the buzzer


To operating the buzzer use the instructions below:

- Press the panic button ②, the buzzer shall be activated.
- Press the panic button again ② to turn the buzzer off.

## 9. Angle adjustment LED light

It is possible to adjust the angle of the LED light with the instructions below:

- Grasp the LED light with your hand.
- Move these downwards or upwards until the desired angle (4 positions: 0° / - 15° / - 30° / -45°).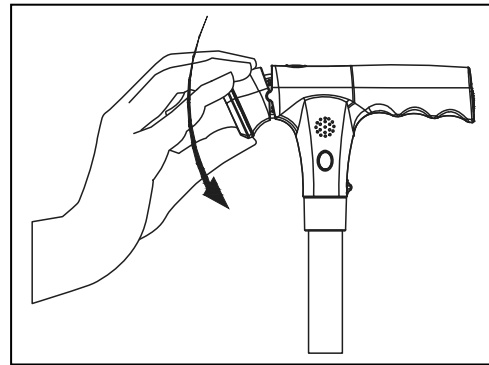


## 10. Height adjustment walking stick

- ⚠ **CAUTION: Risk of falling -**  
Check that the spring pins ② are good clicked back.

Before using the walking stick you must adjust these in the correct height.

Adjust the height of the walking stick according following instructions:

1. Unscrew the anti-noise ring ①.
  2. Press on the spring pin ②.
  3. Pull / push the lower tube ③ until the desired height (10 different positions: increments of 25 mm).
- Ensure that the height of your wrist is at the same height of the handgrip when you leave your arms loose next your body. Ensure that your body is in an upright position.
4. Check that the spring pin ② is good clicked back.
  5. Tighten the anti-noise ring ① again firmly.

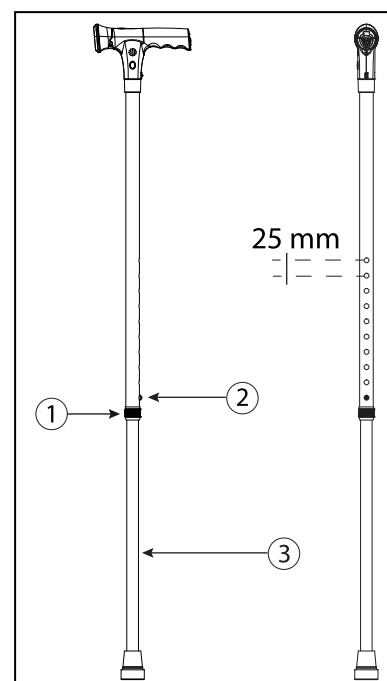## 11. Walking

**⚠ CAUTION: Risk of falling - Do not place the walking stick too far to the front.**

Follow the advice of your doctor or physiotherapist how you must walk with the walking stick. When you have no advice, you can use the instructions below:

- Always walk upright with the walking stick right next to your body.
- Put the walking stick while walking never further than your front leg.
- Be careful with wet/slippery ground.
- Do not place the walking stick in a curve while walking otherwise you can slip out.
- Place the walking stick on the side of your unaffected leg or the side where you need support. Place your affected leg forward simultaneously with the walking stick and then your unaffected leg.

## 12. Walking on stairs

When there is a handrail available, you can use these as support and on the other side you can use the walking stick. Ensure that the handrail is on the healthy side of your body, if possible.

Lean on the handrail and place the walking stick on the first step. Place your unaffected leg on the first step and put your affected leg beside them.

## 13. Safety instructions

- Let someone assist you if this is the first time you are using the walking stick.
- Note that you use your walking stick on a good surface. (Not with too many gravel).
- Be extra careful on a wet / slippery ground.
- Before every usage, check if the walking stick is adjusted in the correct height.
- Check that the spring pins for the height adjustment are good clicked back and the anti noise-ring is firmly secured.
- Do not place the walking stick in a curve or too far from your body to avoid risk of falling.
- Do not use the walking stick as a toy.
- Do not hold the handgrip with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not take too big steps with the walking stick.
- **The max. user weight of the product is 120 kg.**

- Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

## 14. Maintenance

For the maintenance manual of the walking sticks refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com).

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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### **All rights reserved, including translation.**
### Vermeiren Suisse S.A.