Adjustable armrest: B05

Some wheelchairs can be delivered with adjustable armrests: B05. This option give an extra adjustment: "height adjustment armpad".



1. Technical Specifications

Description	Dimensions
Range height armpad	80 mm
Weight	1,40 kg / piece



2. Safety rules

⚠ Check that the height adjustment is firmly secured before using the armrest.

3. Manner of delivery

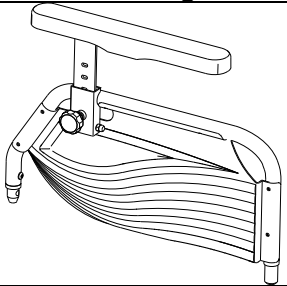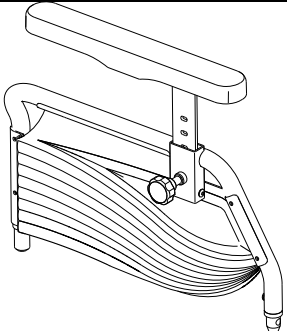Description	Number	Drawing
Adjustable Armrest: B05 - Left	1	
Adjustable Armrest: B05 - Right	1	

4. Assembly / Disassembly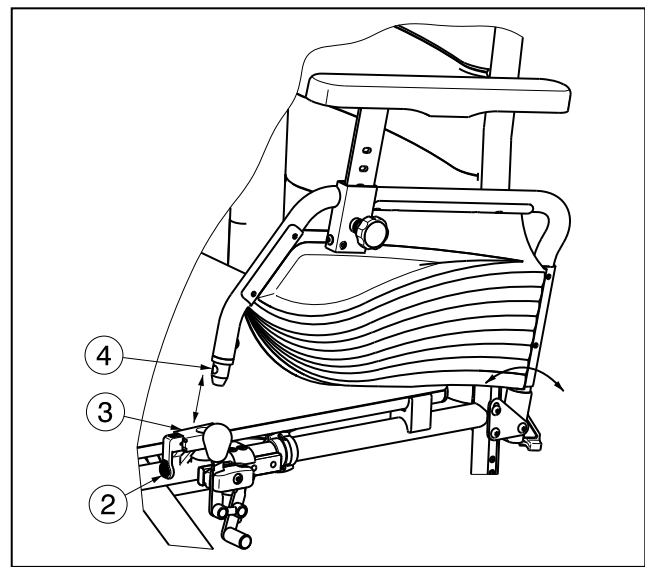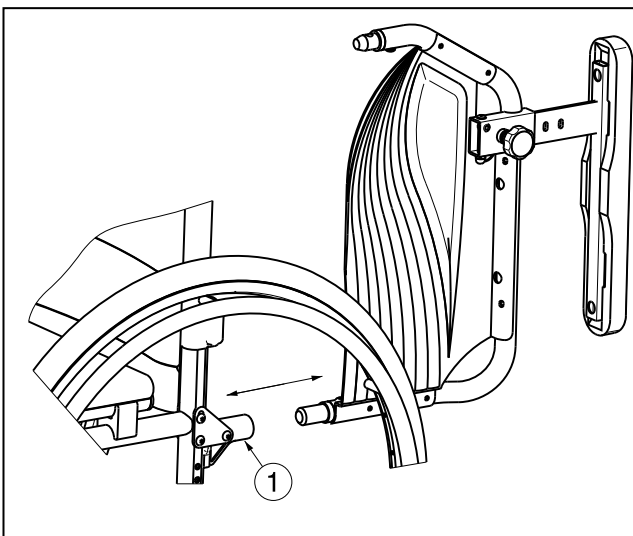


The armrests of the wheelchair can be mounted by following instructions:

1. Mount the rear tube of the armrest in tube hood ①.
2. Make sure the armrest is mounted very well.
3. Fold the armrest forward.
4. Pull the lever ② on tube hood ③ upwards.
5. Click the front tube of the armrest ④ in tube hood ③.

To open and remove the armrest:

1. Pull lever ② and pull the front of the armrest ④ upwards.
2. Fold the armrest backwards.
3. To remove the armrest, pull the rear of the armrest from tube hood ①.

5. Adjusting height of armpad

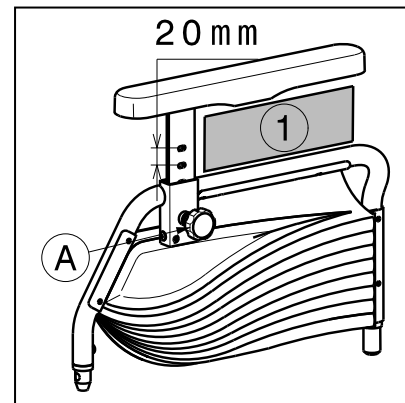The armpad of your wheelchair can be adjusted in height and depth. The height of the armpad can be adjusted in 5 positions (range 80 mm in steps from 20 mm).

Adjust the height of the armpad as follow:

1. Loosen the locking mechanism slightly by turning the star knob ① anti-clockwise.
2. Pull on the locking pin ①.
3. Move the armpad, to adjust the height of the armpad. Do this at the same time when pulling on the locking pin ①.
4. Let go the locking pin ①.
5. Check that the locking pin ① is securely fixed.
6. Retighten the locking mechanism by turning the star knob ① clockwise to reduce the play on the armpad.

Be careful not to put your fingers, buckles, jewelry, between the indicated entrapment zones (Ⓜ).

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Vermeiren Suisse S.A.