


Seat adjustment angles:

A-a = 0deg

A-b = 2deg

A-c = 4deg

A-d = 6 deg

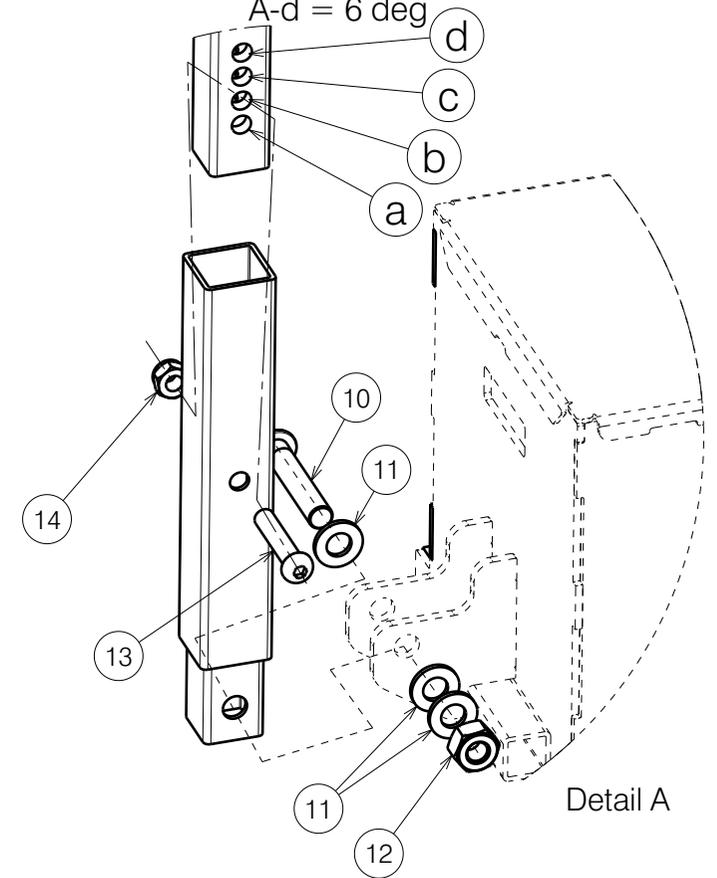

Detail A

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/2

TURIOS MAN SEAT SE30 AND EL BACK SE41

---
Plan No.
EP00625/D1.1

Scale:	1:7	Drafted	Verified	Confirmed
Date	2021-10-27	01/07/2025	01/07/2025	
Initials	MK	AS	MK	



Bill of Material			
No.	Plan	Article	Quantity
1	YP07411 (39)	2776954	1
	P00080268(42)	2689854	1
	YP07412 (45)	2777054	1
	P00080270(48)	2689954	1
2	YP07413 (51)	2777154	1
	YP07414 (39)	2777254	1
	P00080198(42)	2688954	1
	YP07415 (45)	2777354	1
3A	P00080218(48)	2689054	1
	YP07416 (51)	2777454	1
	YP07417 (39)	2777554	1
	P00080264(42)	2689154	1
3B*	YP07418 (45)	2777654	1
	P00080267(48)	2689254	1
	YP07419 (51)	2777754	1
	P00087337(39)	2692554	1
	* P00109052(42)	2910154	1
* P00087866 (45)	2692854	1	
* P00106491(48)	2910254	1	
* P00088204 (51)	2692954	1	
4	YP07421	2777954	1
5	YP07682 (39)	2600854	1
	P00080272(42)	2690454	1
	YP07683 (45)	2600954	1
	P00080293(48)	2690554	1
6	YP07684 (51)	2601054	1
	YP07706 (39)	2779754	1
	P00080301(42)	2690254	1
	YP07707 (45)	2779854	1
7	P00080303(48)	2690354	1
	YP07708 (51)	2779954	1
	YP07709 (39)	2780054	1
	P00080319(42)	2690054	1
8	YP07710 (45)	2780154	1
	P00080302(48)	2690154	1
	YP07711 (51)	2780254	1
	YP07736	27817	1

Bill of Material			
No.	Plan	Article	Quantity
9	YP09294	26433	1
10	M10x55	1998518	2
11	WASHER M10	1001558	10
12	NUT M10	1991858	4
13	M8x40	1008739	1
14	NUT M8	1001632	2
15	M10x45	1908848	2

\* - Not depicted |  
3A - Use with B06 |  
3B - Use with BZ7/8 and BZE |

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/2

---

TURIOS MAN SEAT SE30 AND EL BACK SE41

Plan No.

EP00625/D1.1

Scale: 1:7	Drafted	Verified	Confirmed
Date	2021-10-27	01/07/2025	01/07/2025
Initials	MK	AS	MK

