

9 EC\_A3



5 EC\_COM2



8 EC\_WIRE1  
Total length: 1000



15 EC\_WIRE15  
Total length: 500

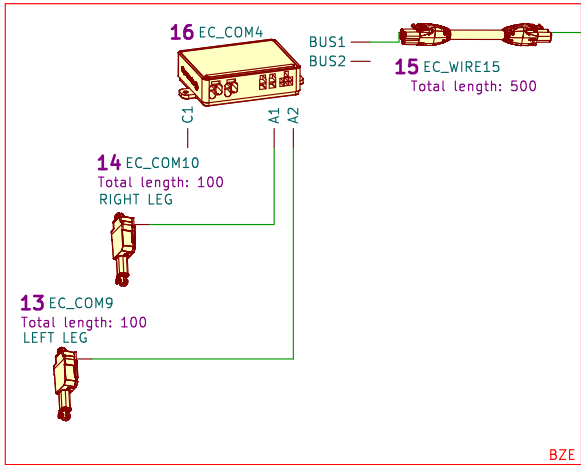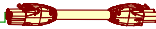

14 EC\_COM10  
Total length: 100  
RIGHT LEG




13 EC\_COM9  
Total length: 100  
LEFT LEG




17 EC\_COM17  
Total length: 100  
TILT  
SE39 SE40



18 EC\_COM18  
Total length: 100  
RECLINE  
SE39 SE41

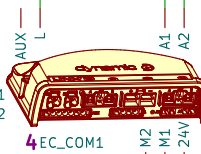


4 EC\_COM1

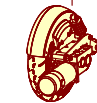

6 EC\_MOT2

7 EC\_MOT1

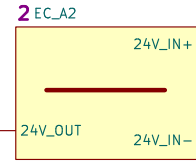

2 EC\_A2

1 EC\_BAT1



1 EC\_BAT2

3 EC\_A1



- BK BLACK
- RD RED
- LBL LIGHT BLUE
- BL BLUE
- LGR LIGHT GREEN
- GR GREEN
- VI VILLET
- OR ORANGE
- YE YELLOW
- WH WHITE
- LRD LIGHT RED
- PK PINK
- GY GRAY
- / AND
- | OR

BZE

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER PROVIDES WRITTEN AUTHORIZATION FROM VERMEIREN GROUP

2992525/C

Plan No.

2992525/C

Id: 1/2

### EC A TURIOS FULL

Size: A4	Drafted	Verified	Conformed
Date:	2025-12-02	2025-12-02	2025-12-02
Initials:	FM	MK	PM



#	Reference	AS	Description	Option	Qty
1	EC_BAT1, EC_BAT2	1917939	EC BAT BS E BATTERIJ 12V DAB12-44EV-HD AGM	44AH	2
		1916730	EC BAT 50AH CPX	PL	2
		1904684	EC BAT 50AH MK	50AH	2
2	EC_A2	2995296	EC A TURIOS BATTERY HARNESS SET 1		1
3	EC_A1	2995225	EC A TURIOS BATTERY FUSE 150A		1
4	EC_COM1	1912294	EC COM DLX-PM60AL-C	6Km/h	1
		1912302	EC COM DLX-PM75AL-C	10Km/h	1
5	EC_COM2	1912301	EC COM DLX-REM210-A	REM210	1
		1912305	EC COM DLX-REM216-A	REM211	1
		1912296	EC COM DLX-REM215-A	REM215	1
		1912305	EC COM DLX-REM216-A	REM216	1
		1913524	EC COM DLX-REM400-B	LCD	1
6	EC_MOT1	1910945	EC_MOT 6Km/h R	6Km/h	1
		1916235	EC_MOT 10Km/h R	10Km/h	1
7	EC_MOT2	1910944	EC_MOT 6Km/h L	6Km/h	1
		1916234	EC_MOT 10Km/h L	10Km/h	1
8	EC_WIRE1	1909022	EC COM GLM-BUS100-A		1
9		2995292	EC A CABLE LINX SE19 SE39	SE19	1
10	EC_A4	2995293	EC A TURIOS LIGHTS E CABLE	SE19	1
11	EC_A5	2995289	EC A TURIOS LIGHTS FRONT	SE19	1
12	EC_A6	2995290	EC A TURIOS LIGHTS REAR	SE19	1
		1914000	EC COM DLX-ACU200-A	SE09	1
		1916653	EC COM DLX-USB02-A	SE06	1
13	EC_COM9	1917021	EC COM LINAKMOTOR BZE	BZE	1
14	EC_COM10	1917021	EC COM LINAKMOTOR BZE	BZE	1
15	EC_WIRE15	1912303	EC COM GLM-BUS050-A	BZE	1
16	EC_COM16	1912304	EC COM DLX-ACT200-A	BZE	1
17	EC_COM17	1908165	EC COM LINAKMOTOR	SE39 SE40	1
18	EC_COM18	1906790	EC COM LINAKMOTOR	SE39 SE41	1

BK BLACK  
RD RED  
LBL LIGHT BLUE  
BL BLUE  
LGR LIGHT GREEN  
GR GREEN  
VI VILLET  
OR ORANGE  
YE YELLOW  
WH WHITE  
LRD LIGHT RED  
PK PINK  
GY GRAY  
/ AND  
I OR

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION FROM VERMEIREN GROUP

2992525/C

## EC A TURIOS FULL

Id: 2/2

Plan No.

2992525/C

Size: A4

Drafted

Verified

Comformed

Date:

2025-12-02

2025-12-02

2025-12-02

Initials:

FM

MK

PM

