# User manual

MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

## Neoflex

### *Neo 30*

**EN**

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Verze: A, 2023-10

**All rights reserved, including translation.**

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*Bedienungshandbuch*

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*Manual de instrucciones*

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This page is intentionally left blank

# Content

<b>Content</b> .....	<b>1</b>
<b>Preface</b> .....	<b>2</b>
<b>1 Your product</b> .....	<b>3</b>
<b>2 Before use</b> .....	<b>4</b>
2.1 Intended use .....	4
2.2 General safety instructions.....	4
2.3 Symbols on the product .....	5
<b>3 Using your Neoflex system</b> .....	<b>6</b>
3.1 Unfastening/Fastening the belts.....	6
<b>4 Maintenance</b> .....	<b>7</b>
4.1 Time of maintenance.....	7
4.2 Instructions for maintenance .....	7
4.3 Expected service life .....	7
4.4 Reuse.....	7
4.5 End of use.....	7
4.6 Warranty.....	7
<b>5 Technical specifications</b> .....	<b>8</b>

## Preface

Congratulations! You are now owner of a Vermeiren Neoflex belt!

Thanks for your trust in the products of Vermeiren. To support you on the use and operating options of your product, this manual is offered. If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. They will be glad to help you on this subject.

### **Important note**

To ensure your safety, and to prolong the lifetime of your product, please take good care of it according to the instructions in this manual.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without being imposed to any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

### **Information available**

On our website <http://www.vermeiren.com/> you will always find the most recent version of following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

This user manual  
For user and specialist dealer


User manual of the wheelchair  
For user and specialist dealer


EC declaration of conformity

# 1 Your product

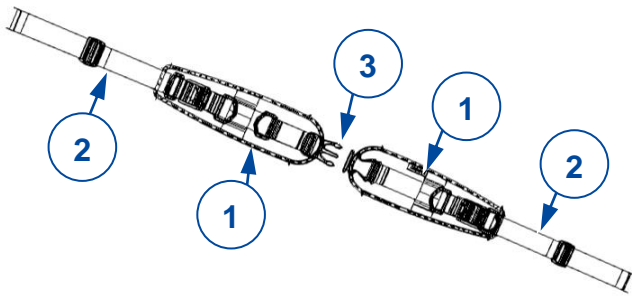
Neo 30.01

Neo 30.02

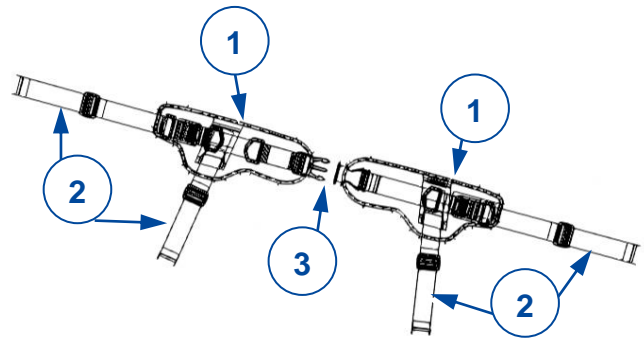Neo 30.03

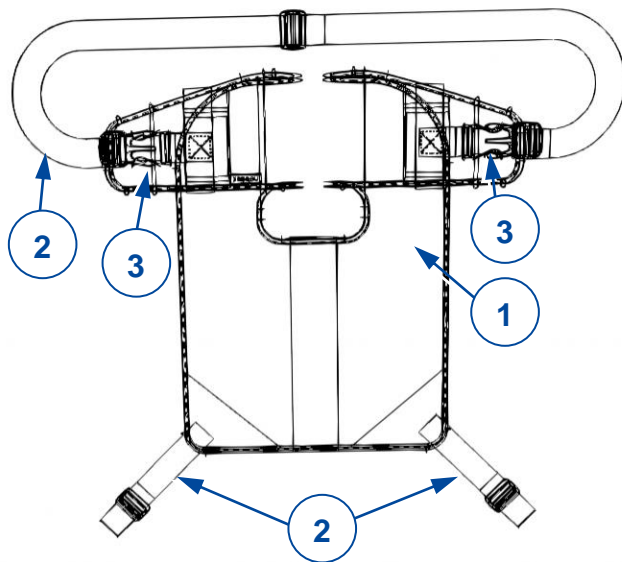1. Padded support
2. Attachment and adjustment belts
3. Buckles

## 2 Before use

### 2.1 Intended use

In this paragraph a brief description of the intended use of your product is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- Indications and contraindications: the Neoflex stabilisation belts are intended for people who require external support while sitting in order to maintain a correct position, or for those who want to stop the process of pathological positioning of the trunk, pelvis or lower limbs. The system should only be used by people with healthy skin, normal breathing, no mental disabilities, and no fractured bones in places where the Neoflex belts are touching the body. Body parts in contact with Neoflex should be covered by clothing. It should not be used for pregnant women.
- The stabilisation belts are intended be used on wheelchairs or other seating systems. They should NOT be used as a restraint or as a seat belt in a moving vehicle.
- Please read all technical details and limits of your Neoflex system in chapter 5.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

### 2.2 General safety instructions

CAUTION

Risk of injuries and/or damage

- Please read and follow the instructions in this manual. Otherwise you may get injured or your product may get damaged.
- **This product should only be installed and fitted to the user by a qualified person.**

Keep the following general warnings in mind during use:

- Only use the attachment systems provided by Vermeiren. Do not structurally modify the device to which Neoflex is attached.
- Before each use, check the product and attachments to the wheelchair/chair to make sure they are undamaged and well secured.
- Only use the Neoflex system over clothing.
- After each use, check the user's skin in the places of contact. Increasing redness and irritation can evolve into more serious injuries such as pressure ulcers.
- In case of damage, do not attempt to repair the product yourself. Contact your specialist dealer.
- Do not modify your product in any way.
- Accidental unfastening of the Neoflex system may cause the user to fall. For users with a cognitive or motion disorder that may lead to accidental release, an attendant or caregiver should be present when using the Neoflex system.
- It is not recommended to use the device when smoking.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For future communication, make sure to keep the LOT number from the packaging.

## 2.3 Symbols on the product

The symbols in following list are applicable to your Neoflex system. Symbols can be found in the relevant ISO standard (ISO 7000, ISO 7001 and IEC 417).

Declaration of conformity

Medical device

Batch code

Do not iron

Machine-washable at 40 degrees and using the delicate programme

Handwash

Do not bleach

Do not tumble dry

Do not dry clean

Manufacturer

Attention: important information

It is advised to read the manual

## 3 Using your Neoflex system

**WARNING**

Risk of injuries

- First read previous chapters and inform yourself about the intended use. Do NOT use your Neoflex system unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.

### 3.1 Unfastening/Fastening the belts

**CAUTION**

Risk of injuries or damage

- The following actions can be done by the user, attendant or caretaker. All other adjustments are done by your specialist dealer.
- Always fasten all buckles when using the device.
- Make sure that the attendant or caregiver is aware of how to unfasten the Neoflex system in case of emergency.
- Make sure that your fingers, clothes, buckles don't get trapped during adjustment.

Depending on the type of Neoflex belt, the support system can be (un)fastened using a number of buckles (see §1).

## 4 Maintenance

### 4.1 Time of maintenance

 CAUTION Risk of injuries and damage

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

#### *Before each use*

Inspect the following points:

- All parts (neoprene fabric, stitching, buckles, straps, mounting system): present, undamaged and showing no excessive wear, no discolouration.
- All parts: clean, see § 4.2.1.

Contact your specialist dealer for possible repairs or part replacements.

#### *If stored*

Make sure that your Neoflex system is stored in a dry environment to prevent mould from growing, or the system from being damaged.

### 4.2 Instructions for maintenance

#### 4.2.1 Cleaning

 CAUTION Risk of damage

- Your product is suitable for cleaning by hand wash, or by using the delicate cleaning programme on your washing machine.
- Do not dry clean, bleach or dye your product.

Minor dirt can be cleaned with a cloth. Thorough cleaning by hand wash or a delicate washing programme is advisable depending on the use frequency and the user's needs.

The upholstery can be washed with lukewarm water (max. 40°C) and a mild soap. Do not use abrasive cleaning agents to clean.

Let the Neoflex system dry thoroughly on a flat surface before using. Do not dry the Neoflex system over intense heat sources (e.g. radiators or stoves) or in direct sunlight.

### 4.3 Expected service life

The average service life of your Neoflex system is 5 years. Depending on the frequency of use and maintenance, the service life of your Neoflex product will increase or decrease.

### 4.4 Reuse

Neoflex products are meant for single patient multiple use only.

### 4.5 End of use

At end of life, you need to dispose of your Neoflex system according to the local environmental legislation.




### 4.6 Warranty

The warranty on this product is subject to the general terms and conditions of each country.

## 5 Technical specifications

The technical details below are only valid for the Neoflex systems at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your system has been modified, damaged, or is severely worn.

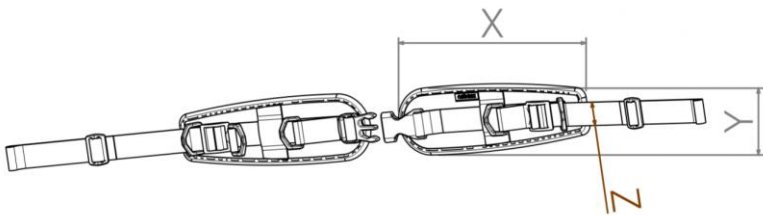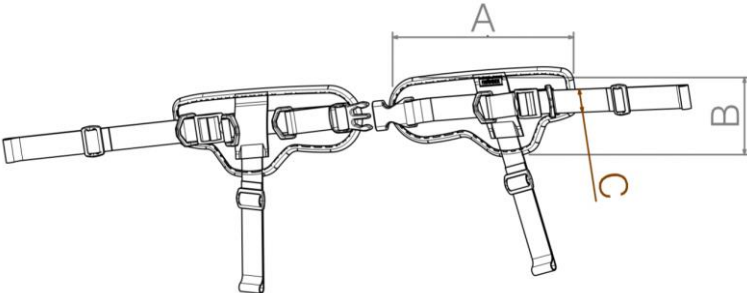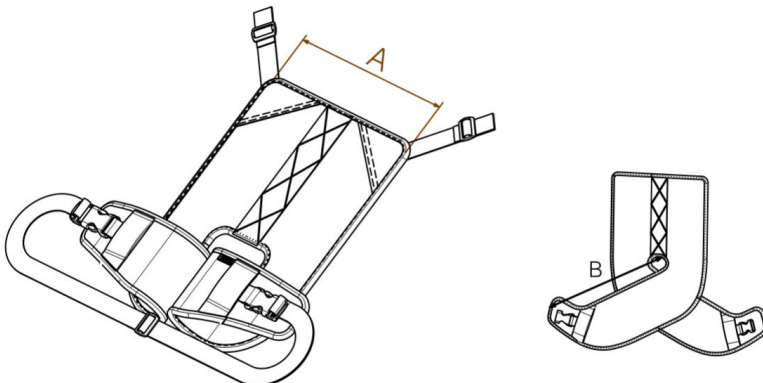Table 1: Technical specifications

Brand	Vermeiren			
Type	Neoflex			
Model	Neo 30			
Description	Dimensions (mm)			
<b>Neo 30.01</b>				
		X	Y	Z
	XS	110	70	25
	S	170	70	25
	M	220	70	25
	L	270	70	25
<b>Neo 30.02</b>				
		A	B	C
	XS	110	70	25
	S	170	90	25
	M	220	90	25
	L	270	90	25
<b>Neo 30.03</b>				
		A	B	
	XS	170	180	
	S	220	280	
	M	270	350	
	L	340	400	
Storage and use temperature	+5 °C ~ +41 °C			
Storage and use humidity	30% ~ 70%			
<b>We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5° .</b>				

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